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## Fabrication method

The cranial orthosis is made from a moulded plaster print of the infant's head. This procedure takes about 30 minutes and is done by our orthotists, in collaboration with the child's parents. The mould is then sent to our central workshop for the preliminary fabrication.

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## Delivery

At the time of delivery, the orthosis is adjusted to optimize comfort and function. Periodic visits, at 2 or 4 week intervals, are necessary in order to ensure a good follow-up of the evolution of the correction process and the child's growth.

During the first week, it is important to check for any red markings on the infant's head. This would indicate abnormal pressure points.

The cranial orthosis should be worn twenty-three (23) hours a day. Generally, this treatment does not need to be repeated. The cranial orthosis treatment can last anywhere from 4 to 8 months, depending on the severity of the Plagiocephaly, its evolution and the child's growth.

Merci à David Dugal et sa maman pour les photos.  
Thanks to David Dugal and his mother for the pictures.

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For our addresses and the different services offered, please contact our nearest branch or visit our website.

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## Cranial Orthosis for Plagiocephaly

For Infants with Positional Plagiocephaly

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**Plagiocephaly is reported in up to 90% of children with congenital Muscular Torticollis.**

American Academy of Orthotists and Prosthetists, 2004

## What is Plagiocephaly?

*Plagiocephaly is a benign deformation of the skull in infants. It arises from one of the following situations:*

- *Improper positioning in the womb (often seen in multiple births)*
- *Presence of congenital torticollis at birth*
- *Prolonged positioning on back due to premature birth or hospitalization.*

*Male infants are more likely to have this condition since they are generally less active in the first few months of their lives.*

*A follow-up in pediatric physiotherapy combined with the use of a cranial orthosis is recommended for a child with congenital torticollis. The child may be referred to a pediatric osteopath, combining three treatments for best results. If the plagiocephaly is severe, the use of a cranial orthosis along with physiotherapy treatments may be recommended even if there is no presence of torticollis. The use of the orthosis will re-model the infant's head.*

*A Plagiocephaly diagnosis is usually established within the 4 to 6 month age range. Due to the rapid growth of the cranium at this age, it is preferable to start treatment with a cranial orthosis as soon as the child is diagnosed.*

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***In a study involving 7609 infants under the age of 8 months, 9.9% had a cranial deformity. After a 3 year follow-up period, 45% of them still had deformities; which would indicate that in almost half of the cases, the problem did not resolve itself with time.***

Boere-Boonekamp MM, van der Linden-Kuiper LT. - Positional preference: prevalence in infants and follow-up after two years, Pediatrics Netherlands, 2001.

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***In a recent study on the early treatment of positioning Plagiocephaly, a group made up of 62 infants was divided into 2 smaller groups. Group 1: 24 infants started treatment before the age of 6 months. Group 2: 38 infants started treatment after the age of 6 months.***

***Significant results were obtained when the 2 groups were compared. Therapy for group 1 was 14 months, compared to 18 months for group 2. Furthermore, the asymmetries for the first group improved by 75.3%, compared to 60.6% for the second group.***

What Is the Optimal Time to Start Helmet Therapy in Positional Plagiocephaly? Timing of Helmet Therapy - Plastic and Reconstructive Surgery - Volume 128, Number 2, August 2011, p 492.

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## What is a cranial orthosis ?

*It is a moulded, padded, plastic helmet specially designed to reshape the cranium during growth. This allows for symmetrical reshaping of the flat parts of the skull.*

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*An untreated Plagiocephaly in the first year of life may result in dental occlusion disorders.*

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## Contraindication

*A cranial orthosis would not be recommended in the case of a child suffering from a rash or uncontrolled eczema, as wearing it would aggravate any skin disorders.*

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*Most insurance companies will defray the cost of a cranial orthosis for Plagiocephaly treatment.*